


SafeMatrix™




Easy as 1, 2, 3

3 Steps to a perfect filling


1 Prepare the SafeMatrix band by pushing the pin in on the side where you want to narrow the band until the pin is flush with the holder. This side must be placed towards the gingiva. The band adapts conically to the tooth anatomy.

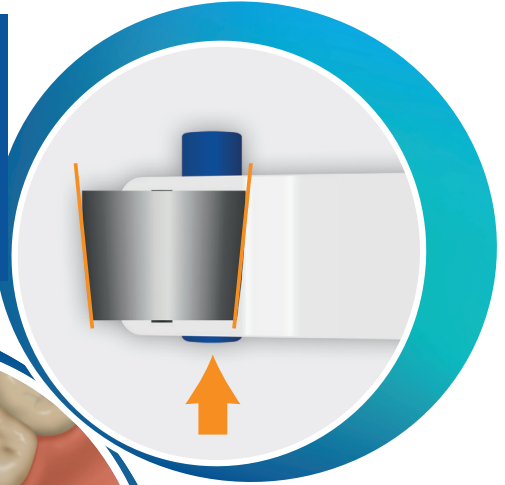

2 Place the matrix band around the tooth and carefully guide it into position. Tighten the band by turning the thumbscrew clockwise. The band can be burnished if necessary. Use wedges if needed.

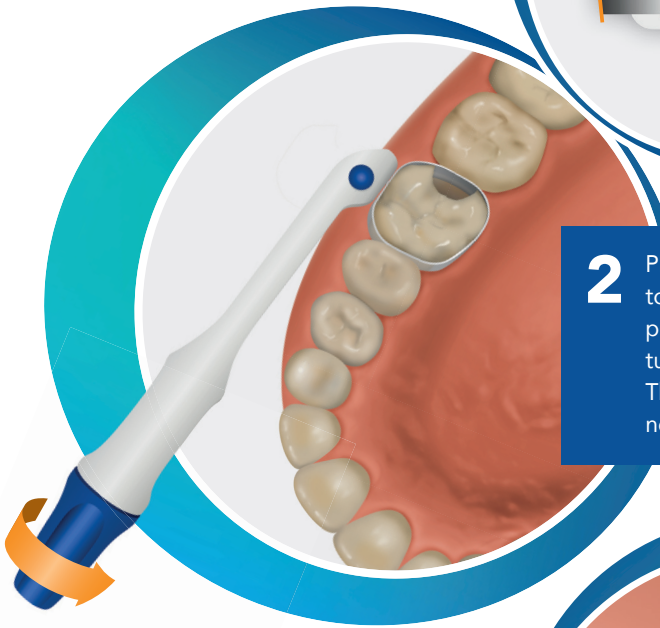

3 Create your restorative filling and cure it as usual. Remove the band by turning the thumbscrew counter-clockwise to loosen it and you're done!

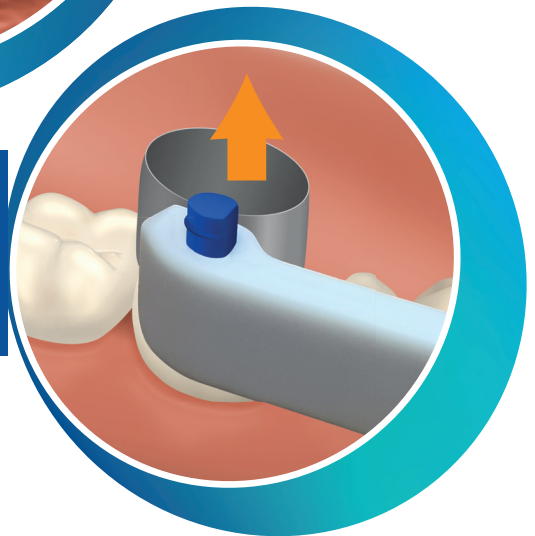

SafeMatrix Benefits

Universal application

SafeMatrix is suitable for all MO/OD and MOD cavities, as well as for cusp build-ups and as a protective shield for the neighbouring tooth during preparation.

Compatibility

May be used with all restorative materials and with wedges and rubber dams.

Optimal fit

Tensioning mechanism and conical adaptation to the tooth ensure tight interproximal contact.

User-friendly design

Narrow neck and rounded edges provide safe and easy access during treatment.

Hygienic

Each matrix is individually packaged. As a single-use product, it does not require reprocessing.

Time-saving


No need for time-consuming assembly of band and holder.

Gentle


Eliminates the risk of injuries to gingiva and cheeks often caused by sharp edges or corroded surfaces on metal holders.

What does "Contour" mean?

The SafeMatrix Contour matrix band is preformed to conform to the shape of the tooth.

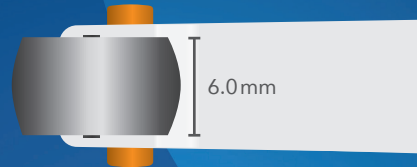

SafeMatrix Contour
Wide 6.0mm

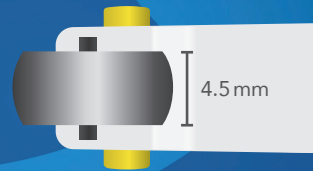

SafeMatrix Contour
Narrow 4.5mm

How to order

+49 21 71 - 70 66 70
@ info@loser.eu



SafeMatrix Contour Wide

- Band width: 6.0 mm
- Preformed
- Content: 50 pcs. (single packed)

REF: 30055



SafeMatrix Contour Narrow

- Band width: 4.5 mm
- Preformed
- Content: 50 pcs. (single packed)

REF: 30060



SafeMatrix Wide

- Band width: 6.0 mm
- Content: 50 pcs. (single packed)

REF: 30045



SafeMatrix Narrow

- Band width: 4.5 mm
- Content: 50 pcs. (single packed)

REF: 30050

The illustrations may show deviations from the original product in terms of colour etc. Our general terms and conditions apply.

